
ADX Collection Card

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1. Let's get organized.

Get familiar with the kit before we get down to business.

The **Blood Test** should contain these items: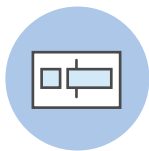

ADX blood collection card (can include one or two strips)

Alcohol pads

Sterile gauze pads

Single use lancets

Adhesive bandages

Blood sample return pouch with dessicant pack

Take the ADX blood collection card and write your **name, date of birth, and collection date in MM/DD/YYYY format** at the bottom of the card where prompted. Now you're ready to start self-testing!

Blood Test - English

2. Now onto the sample collection.

It might look intimidating, but we have faith in you. You got this!


Warm up, prep your test, and wash your hands

Get your blood flowing by drinking plenty of water beforehand! Do a set of jumping jacks, or swing your arms by your side. Place the collection card on a clean and dry surface, making sure to only touch the edges of the card. Then give your hands a good wash with warm water.

Clean your fingertip and prick with lancet


Clean your fingertip with the alcohol pad. Try using the middle or ring finger of your non-dominant hand. Then, take a lancet and twist the cap off. Press the small tip into the side of your finger near the tip, until the needle ejects with a click. Keep your hand below your heart to help the blood flow.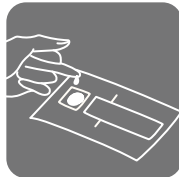

Fill collection card past the first line

Wipe off the first drop of blood with a gauze pad. Place 4-7 large drops of blood in the left square of the card (the top left square if there are two strips in a card), or as many drops as it takes for the blood to migrate past the first line. If the blood does not pass the first line, our lab will not be able to analyze your sample. If you weren't able to fill past the line with one prick, repeat the steps above with a new finger and extra lancet. Don't add additional blood to squares once they have dried--this will invalidate the test!

[Blood Test - English](#)

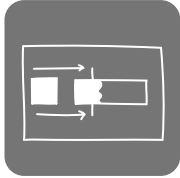Repeat with the second collection strip if applicable, and dry

Your card can have one or two collection strips on it. Repeat the steps above for the second strip if applicable, ensuring again that the blood fills past the first line. Once all strips have been filled, put a bandaid on your finger. Let the card air dry for one hour.

Package sample

Tear open the blood sample return pouch provided. Do not remove any of the contents inside. Place the dried card in the bag and seal. Congrats, you're done! Keep the sample out of extreme temperatures (above 100F or below 32F) before you ship.

3. Pack it up!

Go over the checklist below to ensure your sample is ready to ship.

- Signed your name, date of birth, and collection date on the card
- Filled to the first line on all collection strips on the card
- Placed collection card in the blood sample return pouch
- Put the blood sample return pouch back into the kit box

To return your kit, simply close the box, place the box inside the return envelope, and drop off at your local mail carrier.

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