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# 1. Choose your collection day

Read all instructions carefully and decide the ideal day for you to test. Mark it on your calendar so you don't forget!

## Non-Menstruating People

Collect any day of the month.

## Menstruating People

If testing for Estradiol, Progesterone or Testosterone:

### Regular cycles

Collect on day 19, 20, or 21 of the menstrual cycle (Day 1 = first day of period)

### Irregular cycles

Collect ~5 days before your period starts. Otherwise, collect when you're not bleeding.

For all other tests, you can collect on any day of the month.

**Note: These are recommended collection dates. Collecting outside of these dates will impact the reference ranges used to interpret your results, but will not compromise the sample.**

## 2. Guidelines if you use hormones

**Skip this section if you don't use hormones.**

**Topical:** Don't apply any topical hormones throughout the day of collection. Make sure your last dose is applied at least 12-24 hours before the first collection. Avoid using your bare hands for application for two days before collection (use a glove, or syringe applicator).

**Oral:** If using progesterone, collect saliva 6-10 hours after the last dose. For all other hormones, collect saliva 12-24 hours after the last dose.

**Patches, Vaginal Rings, Pellets or Injectables:** Test half-way between doses. For example, collect 3 days after a once-a-week patch or 2 weeks after a once-a-month injection.

**Troches or Sublingual Therapies:** Talk to your healthcare provider about blood spot testing instead of saliva testing for hormones. If you do move forward with saliva testing, don't use any sublingual hormones during the day of saliva collection. Collect saliva 12-24 hours after the last dose.

**7-Keto DHEA:** Avoid for 3 days prior to collecting. Does not apply to regular DHEA.

**Steroids (e.g. glucocorticoids, hydrocortisone, prednisone):** Talk to your health care provider about appropriate timing.

**Anti-aging creams:** Avoid using for 3 days prior to testing as they may contain undisclosed hormones.

### 3. Prep for collection

Get familiar with the kit before we get down to business.

The **Saliva Test** should contain these items:

Up to four  
collection tubes

The timing of collection will depend on the number of tubes you have:

No. of tubes	Upon waking	Before lunch	Before dinner	Before bed
1 tube				
2 tubes				
4 tubes				

Before every collection, **write your name, date of birth, collection date in MM/DD/YYYY format, and collection time** on the collection tube in the designated area.

Do not eat, drink (except water) or brush your teeth prior to your first collection and for the 2 hours prior to collecting throughout the day.

Saliva Test - English

## 4. Now onto the sample collection!

It might look intimidating, but we have faith in you. You got this!

### Wash your hands and rinse your mouth

Begin collecting your first saliva sample within 30 minutes of waking for the day. Wash hands with soap and water. Then, rinse your mouth with water. Wait 5 minutes before beginning the collection.

### Fill the collection tube with saliva

Take a collection tube, and make sure you have written your name, date of birth, collection date, and collection time in the designated area. Then open the tube to start collecting saliva. Allow saliva to pool in the mouth, then tilt your head forward and gently spit into the tube. Fill the tube until it is **at least  $\frac{3}{4}$  of the way full**, not including bubbles. It may help to imagine your favorite food, or smell some food. When done, cap the tube.

### Store the sample

Make sure the cap is secured tightly on the tube, and that you've indicated collection time on the tube. Place the sample inside the biohazard bag provided, close the bag tightly, and store in the freezer until you mail it.

### **Repeat for additional collection if needed**

Repeat the previous steps for additional collection times if needed (one at night for two tubes total; midday, evening, and night for four tubes total) . Do not eat, drink (except water), or brush your teeth at least two hours before collections.

## **5. Pack it up!**

**Go over the checklist below to ensure your sample is ready to ship.**

- Used all equipment for the Saliva Test to collect sample(s)
- Wrote your name, date of birth, collection date, and collection time on the tube(s)
- Placed sealed tube(s) in the biohazard bag and stored them in the freezer before shipping
- Put bagged sample(s) back into the kit box

**To return your kit, simply close the box, place the box inside the return envelope, and drop off at your local mail carrier.**

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