Fecal Immunochemical Test

**FIT**

**Instructions**

**Instrucciones**

**para FIT**

# 1. Important collection info

**Some conditions may cause an inaccurate result. Do not perform this test if:**

- 💧 You have hemorrhoids that are bleeding.
- 💧 You have blood in your urine. Contact your doctor if this is the case!
- 💧 It is during your period, or within three days before or after.
- 💧 You have bleeding wounds on your hands.
- 💧 Your toilet bowl water is saltwater or rusty.

Note that although you don't have to avoid any foods or medications, eating fruits and vegetables can increase test accuracy!

If you have cleaners or bluing agents in your toilet bowl or tank, remove them and flush the toilet twice before beginning collection.

## 2. Prep for collection

Get familiar with the kit before we get down to business.

The **FIT test** should contain these items:

Collection card

Collection label


Collection envelope


Brush kit

includes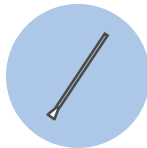

Brushes

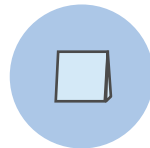Waste bags

Take the provided label and **write your name, date of birth, and collection date in MM/DD/YYYY format** where prompted.

## 3. Now onto the sample collection!

It might look intimidating, but we have faith in you. You got this!

### Prepare for sample collection

When you are about to have a bowel movement, take these instructions, Brush Kit, collection card, and collection label into the bathroom. Flush the toilet **before** your bowel movement. Have the Brush Kit opened so you can readily access the brushes and bags inside.

### Place used toilet paper in waste bag

After your bowel movement, **do not place used toilet paper in the toilet bowl**. Instead, use one of the blue waste bags provided in the Brush Kit. **Do not flush the toilet** after your bowel movement.

### Collect water sample

Take the collection card and lift the flap marked "LIFT HERE FOR SAMPLES". There should be two squares marked "1" and "2". Take one of the blue brushes from the Brush Kit, and gently brush the surface of the stool for 5 seconds. If the stool is loose, stir the water around the stool. Remove the brush from the water and gently shake it to remove excess water and clumps of stool.

### Dab water sample to collection card

Transfer a sample of the **water only** by gently dabbing the brush onto the white square labeled "1" on the collection card for 5 seconds. It's ok if the square is slightly stained. Then, discard the used brush into one of the waste bags and throw it away.

### Repeat sample collection

Using the second blue brush, repeat steps for water sample collection, and transfer a second **water sample** to the test card by dabbing the brush on the white square labeled "2" for 5 seconds. Discard the used brush into the other waste bag and throw it away.

### Seal the collection card and pack it up

Take the collection label, and make sure you have written your name, date of birth, and sample collection date in MM/DD/YYYY format. Peel off the label and use it to reseal the flap on the collection card. Take the collection card and place them into the collection envelope provided. Congrats, you're done!

## 4. Pack it up!

**Go over the checklist below to ensure your sample is ready to ship.**

- Used all equipment for the FIT Test to collect sample(s)
- Wrote your name, date of birth, and collection date on the collection label
- Sealed the collection card with the label, and put it inside the envelope
- Placed envelope into the kit box

**To return your kit, simply close the box, place the box inside the return envelope, and drop off at your local mail carrier. The sample must be returned within 14 days of sample collection for accurate results.**

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